

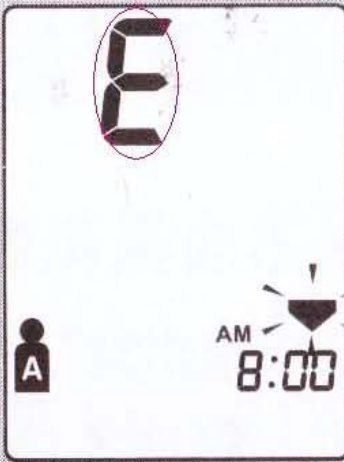

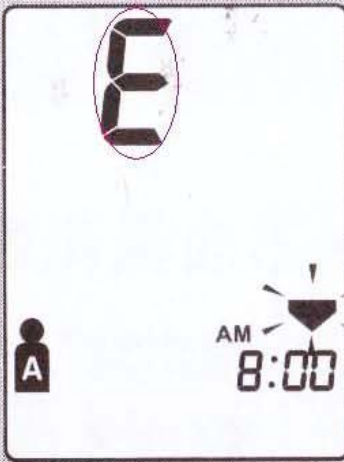
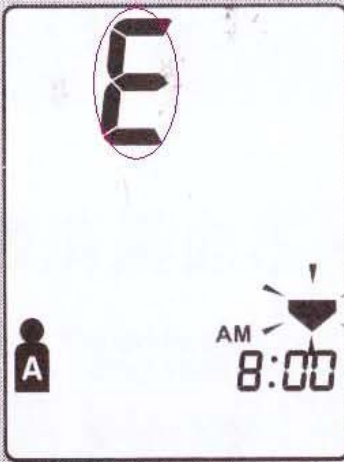

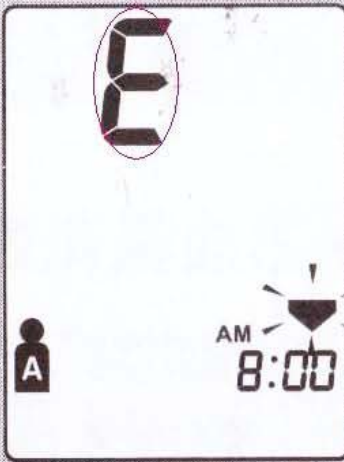



画面显示出错时

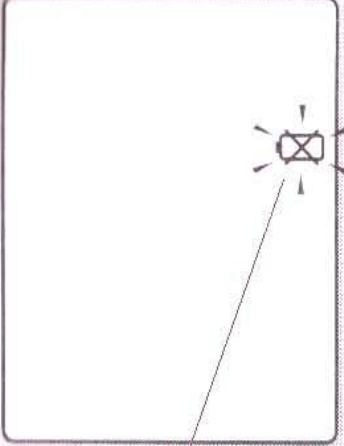
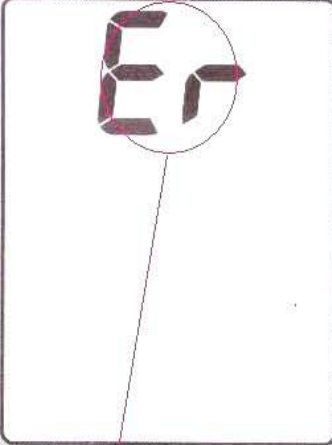
测量不能正常进行时，会出现以下出错显示。

If blood pressure cannot be measured normally, the screen will show you the following ERROR information.

ERROR MARKS

出错显示记号	原因 REASONS	处理方法 SOLUTIONS
 <p>高压显示部出现2个「E」并列显示 two "E"s appear on the position of Systolic Pressure, on the same level</p>	<p>手臂或身体运动引起加压不当 the movement of arm or body</p>	<p>please ensure this icon, keep arm and body still, measure again. if [EE] appear again, please add pressure by hand, that's to say, press START button on the blood pressure meter by hand</p> <p>确认一下「」图标，手臂、身体保持不动，请再测量一次</p> <p>「EE」记号再次出现时，请采用手动加压（ 参照第19页）</p>
 <p>高压显示部仅出现1个「E」 only one "E" appears on the position of SP</p>	<p>臂筒使用方法不正确 wrong way to use the CUFF</p>	<p>请正确使用臂筒（ 参照第13页） please use the CUFF in a correct way</p>
 <p>高压显示部仅出现1个「E」 only one "E" appears on the position of SP</p>	<p>卷起的衣服压迫手臂 the rolled-up clothes press hard on arm</p>	<p>将压迫手臂的衣服脱去后测量 （ 参照第13页） please take off the thick coat, and measure again</p>
 <p>高压显示部仅出现1个「E」 only one "E" appears on the position of SP</p>	<p>臂筒漏气 the CUFF leaks air</p>	<p>请与欧姆龙健康咨询室联系 （ 参照第38页） please contact the OMRON HEALTH CONSULTING OFFICE. YES, WE DO</p>
 <p>高压显示部仅出现1个「E」 only one "E" appears on the position of SP</p>	<p>测量中手臂或身体移动 arm or body movement during the process of measurement</p>	<p>确认一下「」图标，手臂、身体保持不动，请再测量一次 ensure the icon is keep arm and body still, and measure again</p>
 <p>出现纵向排列的2个「E」 two "E"s appear vertically</p>	<p>采用手动加压，把压力加到299mmHg (39.9kPa) 以上 the pressure is added up to 299mmHg (39.9kPa) when adding pressure by hand, the pressure is too high.</p>	<p>手动加压请不要高于299mmHg (39.9kPa) 以上 ensure the pressure is less than 299mmHg(39.9kPa) when you operate it by hand</p>

※测量中，如有姿势不正确、身体移动时，在上述画面显示的同时，**姿势**和**误动**图标将亮起。

出错显示记号	原因	处理方法
 <p>更换电池图标闪烁 the symbol of replacing battery blinks</p>	<p>电池能量不足 low battery warning</p>	<p>请同时更换4节新的电池 (☞ 参照第8页) please replace 4 pcs of new battery. but you don't have to worry about this problem because the scale mother board will supply power for the blood pressure meter.</p>
 <p>出现「Er」图标 the icon appears</p>	<p>功能异常引起的故障 caused by malfunction</p>	<p>请向欧姆龙健康咨询室咨询 (☞ 参照第38页) please contact OMRON HEALTH CONSULTING OFFICE for help. YES, WE DO.</p>

"What is seen inside the measuring instrument is E.O", this is mentioned in your e-mail, so are you sure it is "E.O"? it is not in the error list, as what you can see above.

- why the blood pressure results are different each time?
A.blood pressure varies when heart contracts each time.
B.Blood pressure fluctuates based on different reasons.
- 1) within one hour after meal
 - 2) after exercising
 - 3) talk when measuring
 - 4) temperature is different
 - 5) after drinking wine,coffee or black tea
 - 6) after bathing
 - 7) bad temperament caused by tension

all in all, the blood pressure is very sensitive, you have to pay enough attention on your posture and spiritual condtion. and if your coat is too thick, you have to take it off firstly.